



Find new freedom with Autochair

The Milford Person Lift & Mobile Base



The Milford Person Lift -
safety, comfort and
versatility from one lift.

Key Features

- The aluminium Milford body weighs less than 18 lbs
- Lifts people up to 330 lbs
- Fits most vehicles including cars, vans and mini-vans, crossovers and SUV's
- Lifts you onto your standard car seat
- Emergency up and down buttons
- Shaped extension arm fits around person
- Power is supplied by the vehicles 12 volt battery

Lifting into vehicles

The Milford Person Lift is designed to lift a person, safely and without putting strain on the carer/operator or person being lifted. Manufactured in lightweight aluminium the body of the lift weighs only 18 lbs, while retaining great strength.

Cars – Even the smallest cars can be fitted with the Milford Person Lift (three door cars normally have larger doors than four/five door cars).

Key Benefits

- Can be used on the passenger side of all vehicles. For driver's side, some vehicles may be suitable
- Can be used on back seats in many vans, mini-vans, crossovers and SUV's
- The lift comes complete with sling, electronics and installation kit for the vehicle
- It is guaranteed for 12 months against faulty parts or workmanship
- The Milford Person Lift lifts up and down electrically and the person is moved in and out manually with very little effort
- Heavier people can be lifted with a modified lift – please consult Autochair
- The lift can be set up to suit the vehicle and the person being transferred
- The sling can be fitted around the user while still seated in their wheelchair



1

Fit the sling down your back and under legs then connect



2

Using the hand control, lift person so that their feet fit over the cars sill



3

Maneuver the person into the vehicle



4

Using the hand control lower the person onto vehicle seat



5


Remove the extension arm and main body of lift prior to driving away

Milford Mobile Base

The mobile base gives you the freedom and independence not only in your own home but enables you to take your Milford Person Lift almost anywhere – friends houses, on vacation, or even in a camper or RV.

Key Benefits

- The base is taken apart into 4 easily manageable pieces, each weighing less than 11 lbs and can be assembled in approximately two minutes. Power pack and charger are included
- The Milford can easily be carried in your vehicle and 25+ lifts can be obtained from a full battery

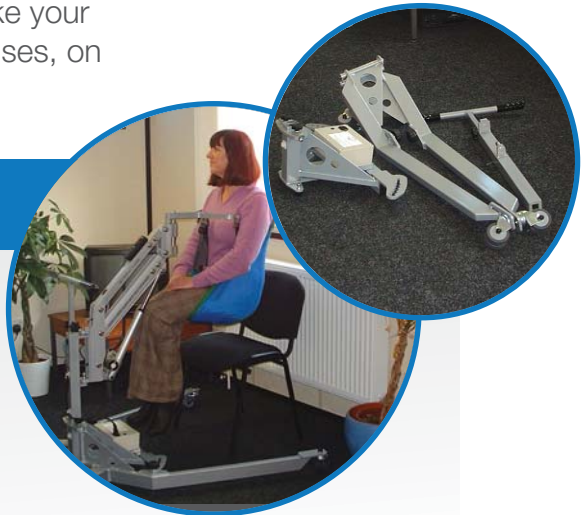


Wall Bracket

For confined areas where a mobile base is not suitable, the wall bracket comes into its own.



Floor Mounted

Special installations can be made to suit your individual requirements such as quick release mounting post in a Camper. Please discuss this with your supplying dealer.

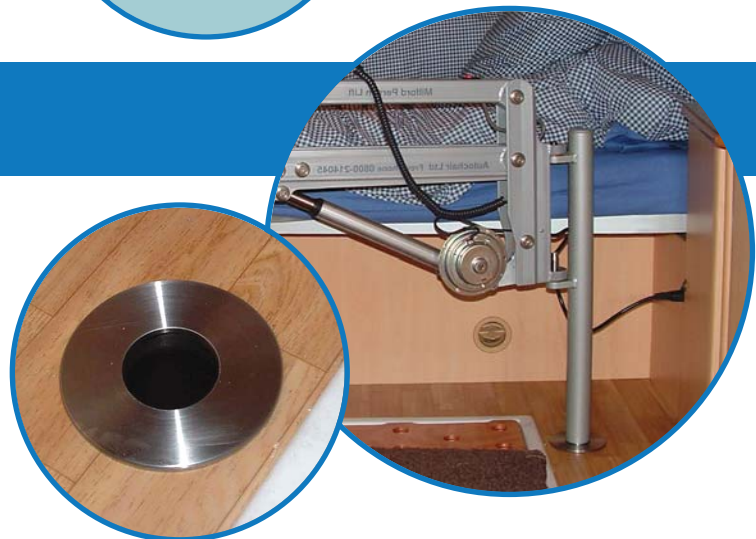

Lift Accessories



Mounting post
in car, lift removed



Lifting arm



Easy to
attach fixings



4 compartment
padded bag, with
carrying straps



Simple hand
control



5 standard
sling sizes
available

Technical Information

- Lifting arm weight: 18 lbs
- Maximum height range of lift: 21 inches
- Fuse rating: 25 amp
- Power supply: 12v
- Slings available: Extra small - Extra large including amputee sling
- Maximum user weight: 330 lbs (heavy duty option available)
- CE Marked
- Padded spreader bar cover, wall bracket extensions and dropped extension arms are also available
- FDA registered



Autochair LLC
1168 North Main Street
Lillington
North Carolina, 27546
Tel 1 (910) 814-1350
Fax 1 (910) 814-1352
info@autochair.com
www.autochair.com

Autochair have over 30 years experience in the design and manufacture of vehicle adaptations to assist people with disabilities and reduced mobility Wheelchair Lifts

- Scooter Lifts
- Swivel Seats
- People Lifts
- Wheelchair Carriers
- SMARTDRIVING CONTROLS

Autochair reserve the right to change product specifications or appearance without prior notice through product development and improvement. We aim to ensure our customers are aware of this at the time of ordering.